## Instructions for Use: Family of Vitrectomy Lenses and Products

**Description:** Family of Vitrectomy Lenses and Products includes these listed single use devices for use during ophthalmic surgery. These are available either packaged together or separately.

REF	Product Description
K30-1440	<b>Plano Lens</b> provides a view of the retina. 36° field of view with image magnification of 1.02X.
K30-1435	<b>Magnifier Lens</b> provides a view of the retina. 30° field of view with image magnification of 1.20X.
K30-1430	<b>Bi-Concave Lens</b> provides a view of the retina. 25° field of view with image magnification of 0.80X.
K30-1420	<b>20 Degree Prism Lens</b> for peripheral imaging of the retina. 36° field of view and image magnification of 1.02X.
K30-1425	<b>30 Degree Prism Lens</b> for far peripheral imaging of the retina. 36° field of view with 1.02X image magnification.
K30-1530	<b>Suture Ring</b> provides a stable platform to hold any of the vitreoretinal lenses.
K30-1535	<b>Suture Ring with tabs</b> provides a stable platform to hold any of the vitreoretinal lenses.
K30-1505	<b>Forceps</b> allow manipulation and placement of any of the vitreoretinal lenses.
K30-1450	<b>Surgical Wide Field Lens</b> produces an indirect wide field image of the retina. Image magnification of 0.42X and 155° field of view.
K30-1520	<b>Surgical Wide Field Lens with Handle:</b> the Surgical Wide Field Lens assembled with a handle.

**Intended Use:** The Family of Vitrectomy Lenses and Products allows visualization of the ocular fundus, vitreous and retinal structures during vitrectomy surgery.

**Usage Instructions:**

- Review all information in these Instructions for Use carefully before use.
- The lenses and products are for use by qualified Ophthalmologists and Surgeons only.
- Inspect packaging before use to ensure that sterile barrier is intact.

**Caution: Do not use devices if not provided in original packaging, or if opened or damaged, as product sterility could possibly be compromised.**

- Remove device from package and inspect for damage (such as scratches or sharp edges) before use.

**Caution: Damaged devices should not be used, in order to avoid any harm to the ocular surface of the patient.**

- Follow standard protocol for your facility when using these devices.
- Dispose of properly after each use. These are single use, disposable devices and may not be re-used.

**Warnings, Notes, and other Information to ensure Safety:**

The Vitrectomy Lenses and Products are primarily used in hospitals or surgical centers. They are to be used only by persons appropriately trained and qualified in the clinical and/or surgical use of such devices.

**Caution: US Federal law restricts this device to sale by or on the order of a physician.**

**Caution: Procedures should be conducted with new devices only as reuse could transmit infection or disease between patients.**

Failure to comply with instructions or other information provided could result in injury to the patient.

Assessment by trained medical or otherwise qualified personnel is necessary and due care / caution is to be exercised in cases of infection or the eye or ocular injury.

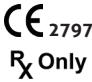
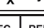
**Specifications / Manufacturer's Information:**

Devices are supplied sterile, via Ethylene Oxide.

Transport / storage / usage conditions: -20 to +60°C (-4 to +140°F)

Any alteration or misuse of the lenses voids the product warranty.

Katena Products, Inc reserves the right to update or alter the technical specifications at any time.

	<b>Symbols Used in Labeling:</b>	
		Caution, consult Instructions for Use
		Sterilized using Ethylene Oxide
		Do not use if package is damaged
		Do not reuse
		Do not resterilize
		Manufacturer
		For prescription use only
		European Authorized Representative
		"Use by" (expiration) date
	Refer to Instructions for Use	

**Katena Products, Inc.**  
6 Campus Drive, Suite 310  
Parsippany, NJ, 07054 USA  
Phone: +1-973-989-1600  
info@katena.com

**European Representative:**  
Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

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