katena (EN)	
Instructies for Use: Reusable Contact Lenses	
REF	Product Description
K31-1002	3 Mirror Diamond Lens: For examination of the ocular fundus and iridocorneal angle. The central axis has a 30° field of view. The mirrors are 75°, 67°, and 59°.
K31-1005	4 Mirror Diamond Lens: For examination of the structures in the anterior segment including the angle and the base of the iris, primarily for glaucoma diagnosis.
K31-1330	Retina 200 Diamond Lens: for diagnosis and treatment of the posterior pole, image magnification: 0.6X

Intended Use: Diagnostic and therapeutic contact lenses used in the examination of the eye fundus, retina and irido-corneal and vitreous bodies and for the laser therapy of intraocular abnormalities.

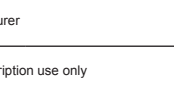
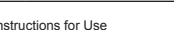
Usage Instructions:

- Review all information in these instructions for Use carefully before use.
- The lenses are for use by qualified Ophthalmologists only.
- Follow standard protocol for your facility when using these devices.
- Visually inspect the contact surface of the lens before use. Do not use the lens if damage is visible on the contact surface, such as rough surface or sharp edge.
- For safe use, the lenses must be cleaned and disinfected before initial use and between uses, as follows:
 - Do not allow the lens to dry after use . Immediately decontaminate the lens after use. If this is not possible, keep it in a container of water for irrigation or sterile normal saline, until it can be decontaminated.
 - Rinse in water for irrigation or sterile normal saline for at least 30 seconds.
 - Clean by rubbing with liquid soap or detergent.
 - Rinse with sterile water and shake off excess water.
 - Disinfect using sodium hypochlorite 1% (10,000 ppm of available chlorine) for 10 minutes. Ensure the entire contact surface is exposed to the solution, with no air bubbles trapped inside. Do not immerse the holding ring in the solution as discoloration of the finish may occur, any such discoloring will not affect the optical performance.
 - Rinse in water for irrigation or sterile normal saline for at least 10 minutes with 3 changes of water/saline.
 - Shake off excess water, then dry with tissue.
 - Store the lens dry, if the lens is not to be re-used immediately.

Warnings, notes, and other information to ensure safety:
Caution: US Federal law restricts this device to sale by or on the order of a physician.
Caution: Transmission of infection and/or ocular injury may occur if cleaning and disinfection are not performed correctly.
Failure to comply with instructions or other information provided could result in injury to the patient.
Assessment by trained medical or otherwise qualified personnel is necessary and due care / caution is to be exercised in cases of infection of the eye or ocular injury. Do not activate the lens; heat sterilization will damage the lens.

Specifications / Manufacturer's Information:

Transport / storage / usage conditions: -20 to +60°C (-4 to +140°F)
Any alteration / modification of the lenses voids the product warranty.
Katena Products, Inc. reserves the right to update or alter the technical specifications at any time.

Symbols Used in Labeling:	 Non-Sterile	 Manufacturer
	 For prescription use only	 European Authorized Representative
	 Refer to Instructions for Use	
	Katena Products, Inc. 4 Stewart Court Denville, NJ 07834, USA Phone: + 1-973-989-1600 info@katena.com	European Representative: Emergo Europe Prinsesgracht 20 2514 AP The Hague The Netherlands

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Katena Products, Inc. diğelledi dđna teknik özellikleri güncellemeye ya da değırtirme hakkını saklı tutar.